



Quick Guide for CAS Software Update

Requirements

- Already have installed any version Cassini software, 3.5.0 or older versions
- Latest software release package:
 - **Cassini-v3.5.1.5301.exe**
 - **LENSAR_LibraryInstaller-v1.0.2.1.exe**
 - **CassiniDICOMInterfaceSoftware-v1.2.0.3770-x86.exe**
- Administrative rights on the target system

Installation steps

1. Preparation:
 - a. Locate and prepare your installers
 - b. Ensure you have administrative rights on the target system
2. Copy **Cassini-v3.5.1.5301.exe** and **LENSAR_LibraryInstaller-v1.0.2.1.exe** to a target system. Make sure you copy them both to the same location.
3. Double-click on **Cassini-v3.5.1.5301.exe**
4. Click on **YES** to accept the User Account Control (UAC) prompt and go through the Cassini Setup Wizard steps, see below.

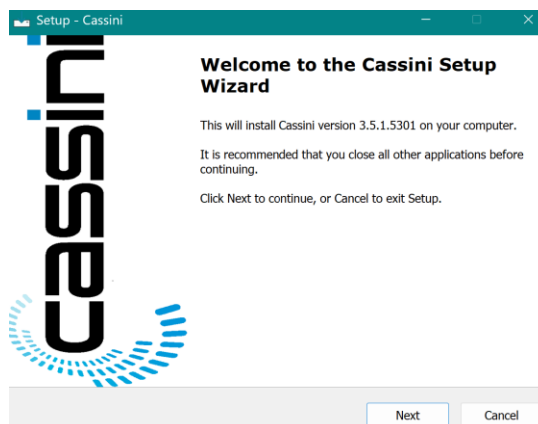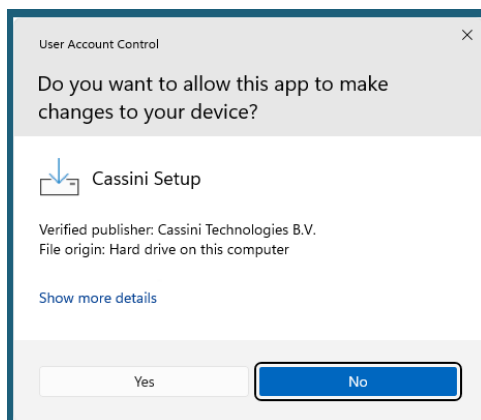

5. At the end of the installation uEye drivers' installation wizard will appear. Go through these steps as well.
6. Once uEye driver has finished installing, the **LENSAR_LibraryInstaller-v1.0.2.1.exe** will automatically launch its installer. Please allow the installer to finalize, then click **Finish**.
7. If necessary, install **CassiniDICOMInterfaceSoftware-v1.2.0.3770-x86.exe**
8. Start the Cassini application.

Note: If you do NOT have the latest release software version listed, please contact us.