


Quick Guide for CGS Software Installation

Requirements

- Software release version:
 - **CassiniGuidance-v1.2.1.3039.exe**
 - **CassiniGuidanceLanguagePack-6.0.exe**
- Administrative rights on the target system

Installation steps

1. Preparation:
 - a. Locate and prepare your installers
 - b. Ensure you have administrative rights on the target system
2. Double-click on **CassiniGuidance-v1.2.1.3039.exe**
3. Click on **YES** to accept the User Account Control (UAC) prompt and go through the Cassini Setup Wizard steps, see below.

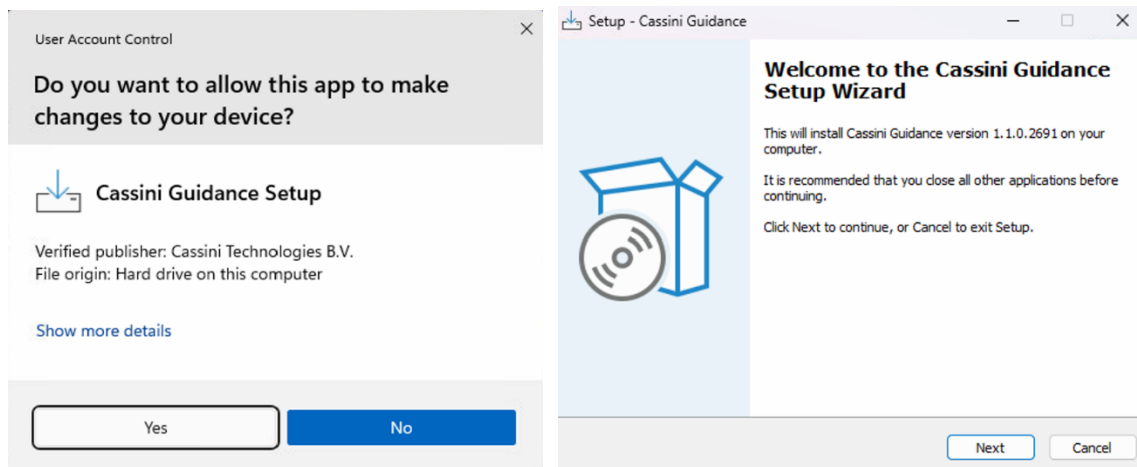

4. The installer will install the IFU languages as part of it.
5. Start the Cassini Guidance System application.

Note: *If you do NOT have the latest release software version listed, please contact us.*