


# Quick Guide for CGS Software Update

## Requirements

- Already have installed any version Cassini Guidance software, v1.0.1.1661 or older versions
- Latest software release version:
  - **CassiniGuidance-v1.1.0.2691.exe**
- Administrative rights on the target system

## Installation steps

1. Preparation:
  - a. Locate and prepare your installers
  - b. Ensure you have administrative rights on the target system
2. Double-click on **CassiniGuidance-v1.1.0.2691.exe**
3. Click on **YES** to accept the User Account Control (UAC) prompt and go through the Cassini Setup Wizard steps, see below.

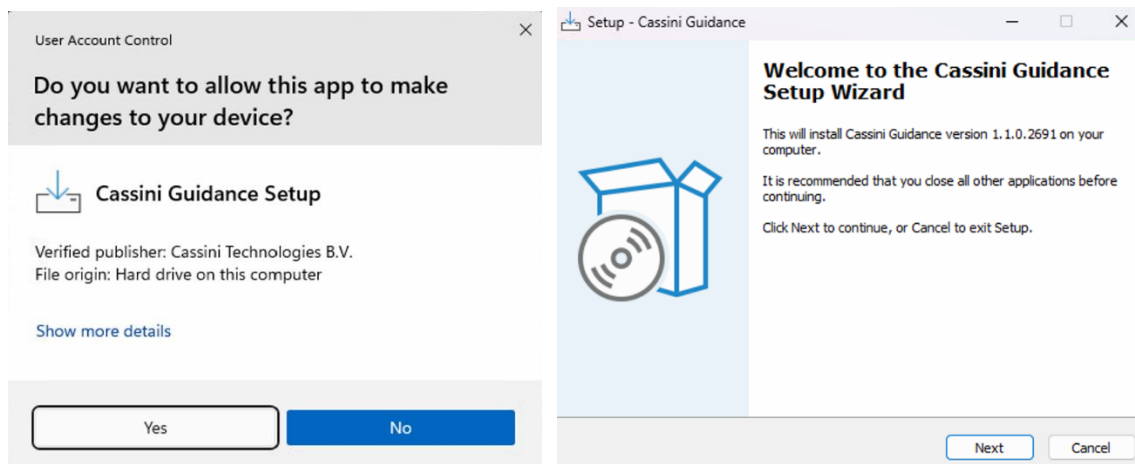

4. Start the Cassini Guidance System application.

**Note:** If you do NOT have the latest release software version listed, please contact us.